



Immedia 4WayGlide System

IN-BED SYSTEM FOR USERS WITH LIMITED MOBILITY

The Immedia 4WayGlide System is an in-bed system for severely immobile patients. It is ideal for patients with reduced mobility or with pain or decubitus issues. The main advantage with this in-bed system is that it can remain in bed at all times. No application or removal of the slide sheets is necessary. Our 4WayGlide System always consists of two parts: a nylon sheet attached to the mattress and a Glide Mattress. The combination of the underside of the Glide Mattress and the nylon sheet provides an extremely low friction surface for movement in all directions, which facilitates positioning and turning, whether manual or with support of a hoist.



The top mattress of the 4WayGlide is equipped with handles that can be attached to a hoist to turn heavy or passive patients.

INTENDED USE

- In-bed system, user with reduced mobility
- In-bed system, heavy user or user with pain or pressure ulcer
- Turning
- Positioning
- Support horizontal transfer
- Can be used with support of a hoist

The Lock On Sheet System (LPL)

A major challenge with an in-bed system is to prevent the patient from unintentionally sliding down, especially when the bed is profiled. The Immedia 4WayGlide System has the unique Lock On Sheet System. This means that the extremely low friction material around the seat easily can be removed to prevent the patient from sliding down. Moreover you can easily put it back on when she has to be repositioned.



1. With the unique Lock on Sheet System, the low friction material can be removed and allow the patient to sit comfortably without sliding down.



2. When the patient has to be repositioned, deactivate the Lock On Sheet System to enable low friction under the seat.



3. The Immedia 4WayGlide System can now be used to reposition the patient higher in bed.



4. Sideways repositioning is also easily done with one or two caregivers.

TIP!

Low friction material allows the upper body to slide when the head end of the bed is raised and shearing is avoided.

Options for the 4WayGlide System


The 4WayGlide System consists of two parts: a nylon sheet and a Glide Mattress. The nylon sheet is attached to the mattress and the Glide Mattress is placed on the nylon sheet.



Immedia
4WayGlide

Standard configuration of the 4WayGlide System

The standard nylon sheet has a unique Lock On Sheet function and non-slip material along the bedside.

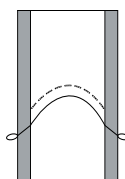

IM85/200LPLNS


The standard Glide Mattress has a cotton/polyester upper. Handles on the inside improve grip and allows attachment to a hoist.



IM140/200LPL


Optional Nylon Sheet

For improved flexibility, the Nylon Sheet has the Lock On Sheet function in three areas. It also features non-slip material along the bedside.

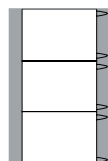

IM85/2003LPLNS


This Nylon Sheet features Lock On functionality. Non-slip material along the bedside is removed to allow transfer from the bed to another supine surface.

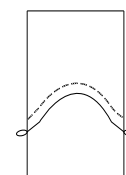

IM85/2003LPL

Optional Glide Mattress

The Glide Mattress has a polyurethane topside. Handles on the inside improve the hand grip and allow attachment to a hoist. The surface is wipe-clean.

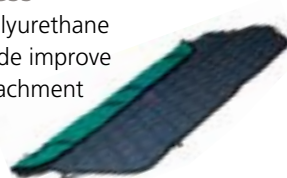

IM140/200LPLG

The TwinSheet Glide Mattress features an integrated incontinence pad. Handles on the inside improve hand grip and allow attachment to a hoist. It is divisible for improved personal hygiene (for further details see page 14).



IM4315

The Immedia 4WayGlide Maxi is a alternative for large and heavy patients.

DESCRIPTION	WIDTH X LENGTH X HEIGHT		ITEM NO.	SPECIFICATIONS
IMMEDIA 4WAYGLIDE LPL	CM	INCH		GLIDE MATTRESS, LPL AND MAXI TOP: Polyester/cotton or polyurethane INSIDE: Polyester, net reinforcement (only Maxi) Underside: Nylon, treated for antistatic HANDLES: Polyester PADDING: Polyester NYLON SHEET: Nylon LOCK: Nylon with PU-coating
~ Glide Mattress, cotton/polyester	140 x 200	55.1 x 78.7	IM140/200LPL	
~ Glide Mattress, polyurethane	140 x 200	55.1 x 78.7	IM140/200LPLG	
~ Nylon Sheet with NS-lock and NS-sides	85 x 200	33.5 x 78.7	IM85/200LPLNS	
~ Nylon Sheet with NS-lock	85 x 200	33.5 x 78.7	IM85/200LPL	
~ Nylon Sheet - 3xNS-lock and NS-sides	85 x 200	33.5 x 78.7	IM85/2003LPL	
IMMEDIA 4WAYGLIDE LPL MAXI				
~ Glide Mattress, cotton/polyester	180 x 210	70.9 x 82.7	IM180/200LPL	
~ Glide Mattress, polyurethane	180 x 200	70.9 x 78.7	IM180/200GLPL	
~ Nylon Sheet - NS-lock and NS-sides	110 x 200	43.3 x 78.7	IM110/200LPLNS	
ACCESSORIES – POSITIONING WEDGE				
~ With non-slip cover	30 x 20 x 53	11.8 x 7.9	IM53NS	
~ With non-slip cover	30 x 20 x 80	11.8 x 7.9 x 30.5	IM53/80NS	
~ Cover PU/cotton, white	140 x 220	55.1 x 82.7	IM140/200PU	
SET – IMMEDIA 4WAYGLIDE LPL				
~ Standard Set – includes IM140/200LPL + IM85/200LPLNS			IM4309	
~ 3-Lock Set – includes IM140/200LPL + IM85/2003LPLNS			IM4300	
WASHING INSTRUCTIONS				
Can be wiped with disinfectant (polyurethane top). Do not use fabric softener or bleach.				
			LPL LPL Maxi	