

Etac Hi-Loo Fixed

Hi-Loo with fixed mounting is a discreet toilet seat raiser without arm supports, that fits most porcelains and is attached firmly and securely to the toilet. The ergonomically designed seat has recesses at the back and front, providing convenient intimate hygiene.

Hi-Loo fixed is available in two heights.



Etac Hi-Loo with arm supports

Hi-Loo toilet seat raiser with arm supports has fixed mounting that fits most porcelains and is attached firmly and securely to the toilet. It has a detachable seat unit with recesses both at front and rear.

The arm supports are foldable but can be permanently fixed in the folded down position. They are detachable and Hi-Loo can be used with only one arm support if space is scarce.



Hi-Loo with arm supports fits most porcelains and is easy to clean.



Easy to detach

The snap locks make it easy to detach the seat unit for cleaning.



Hi-Loo Angled

Hi-Loo Angled is recommended when knee or hip mobility is restricted.

Accessory



Brackets, pair Hi-Loo, Compl.
80301030 Provides extra stability if required for side transfers.

Description

Hi-Loo with arm supports, fixed 6 cm (2½") 80301316-2
Hi-Loo with arm supports, fixed 10 cm(4") 80301317-2
Hi-Loo with arm supports, angled 80301318-2
Hi-Loo fixed, no arm support, 6 cm (2½") 80301106
Hi-Loo fixed, no arm support, 10 cm (4") 80301107

Dimensions

Arm support height from top of seat: 23 cm (9")
Width between arm supports: 49 cm (19¼")
Overall width incl. arm supports: 60 cm (23½")
Hi-Loo Angled:
Rear height: 10 cm (4") Front height 6 cm(2½").

Material

Seat unit: polypropylene
Arm supports: Polypropylene, powder coated steel tubing
Mountings: aluminium

Weight

Hi-Loo with arm supports:
6 cm: 4.9 kg (10.8 lbs)
10 cm: 5.8 kg (12.8 lbs)
Hi-Loo with fixed mounting:
6 cm: 2.4 kg (5.3 lbs)
10 cm: 3 kg (22 lbs)
Angled arm supports: 5.5 kg (12.1 lbs)



Clean the product with a solvent-free cleaning agent (pH level 6–9), or with a 70% disinfectant solution. Can be decontaminated at max. 85°C / 185°F if so required.

